

MagnetOs™

Flex Matrix Surgical Preparation Guide

MagnetOs Flex Matrix stays strong yet flexible even when wet: so you can tear it, mold it, fold it to fill bony spaces any way you need.



Store product at room temperature until needed



1 Open the inner (protective) packaging



2 Place the inner packaging on a sterile field and hydrate MagnetOs Flex Matrix with BMA



3 Tear and shape hydrated MagnetOs Flex Matrix if desired by hand, or forceps



4 Add autograft to the hydrated MagnetOs Flex Matrix



5 Mix/mold autograft and MagnetOs Flex Matrix



6 Mold to shape MagnetOs Flex Matrix before implantation

For important safety and intended use information please refer to the Instructions for Use for a full list of indications, contraindications, precautions, and warnings.

Manufactured by Kuros Biosciences BV
Prof Bronkhorstlaan 10, Building 48, 3723 MB Bilthoven, The Netherlands.
kurosbio.com

MagnetOs is a trademark of Kuros Biosciences.

